

# ProE-Vac™

## Liquid Evacuation System Cleaner



### *Non-Foaming to Protect Pumps.*

ProE-Vac is a non-foaming, biodegradable evacuation system cleaner that leaves a fresh mint scent. ProE-Vac's multi-action formula cleans and deodorizes the entire evacuation system.



#### **No Foam:**

- Protects evacuation systems and helps to extend life.

#### **Effective, Yet Gentle:**

- Regular use keeps tubing, piping and pumps free-flowing.
- Low pH ensures complete penetration through soil, yet is gentle to the evacuation system.

#### **Quick Action:**

- Rapid cleaning of calcium-based materials, saliva, gels and blood.

#### **Safe for All Evacuation Systems:**

- Tested on metals and plastics commonly found in evacuation systems.

#### **Two Easy Packaging Options:**

- Convenient 1 oz. unit dose tubes.
- Economical gallon with a 1 oz. pump (1 gallon contains 128 uses).
- Both mix easily in the heavy-duty dispensing container.



*Tidy Tote™: Convenient closed container for transporting solution between dental units. Special cap included is an adapter for both high volume and saliva ejector unit lines.*

<b>ProE-Vac Description</b>	<b>Case Qty</b>	<b>Order#</b>
1 oz. Unit Dose - 24/box	6	PVACU24
1 Gallon with Pump	4	PVAC128
Tidy Tote - 1 Gallon Empty Dispenser	1	PVACB

# ProE-Vac™

## Liquid Evacuation System Cleaner

### Description:

ProE-Vac is a powerful, non-foaming, concentrated biodegradable evacuation system cleaner that leaves a fresh mint scent.

### Intended Use:

ProE-Vac may be used to clean and deodorize the entire evacuation system, including the high-volume evacuator (HVE), saliva ejector and cuspidor.

### Product Specifications:

Active Ingredients: Phosphoric Acid CAS#7664-38-2  
Glycolic Acid CAS#79-14-1  
pH: Less than 1.5.  
Odor: Mint.  
Color: Pink.  
Shelf Life: 3 years from date of manufacture.

### Product Capabilities:

ProE-Vac is a multi-action formula that cleans and deodorizes the entire evacuation system. The highly effective, non-foaming detergent action reduces offensive odors. The superior surfactant leaves no film or residue. Handy unit dose size makes mixing to correct dilution even easier. One ounce (one unit dose packet) diluted with one quart water treats one dental unit (2 - 3 evacuation lines). Use four ounces (four unit dose packets) per gallon of water.

Regular use of ProE-Vac protects and extends the life of the entire evacuation system and reduces the possibility of system failure. ProE-Vac keeps tubing, piping and pumps free flowing. ProE-Vac's low pH ensures complete penetration through soil, yet is gentle to the evacuation system and its components.

### Use Precautions:

Harmful if swallowed. Do not get in eyes, on skin, or on clothing. Do not store product on counters. Spills may discolor countertops.

---

### Accessory Products:

Other products available from Certol include:



**ProSpray™ wipes**  
Surface Disinfectant/  
Cleaner Towelettes



**Good Vibrations™**  
Ultrasonic Cleaner  
Solution



**ProEZ 2™**  
Dual Enzymatic Detergent



**ProBarrier™**  
**dispo chain™**  
Disposable Bib Chain