

ProEZ AW Quad™

Concentrated Quadruple Enzyme Automatic Washer Detergent



Powerful, Non-Foaming, Four Enzyme Cleaning Formula.

ProEZ AW Quad is a premium, neutral pH, four enzyme instrument cleaning detergent specifically designed for use in high impingement systems. Enzymes and surfactants work as a team to handle tough cleaning challenges.

- Highly concentrated.
- Biodegradable.
- Anti-corrosive agents.
- Instrument brightener.
- Safe for all surgical instruments and scopes.
- Indicated for use in the cleaning cycle of automated scope reprocessors.

Four Powerful Cleaning Enzymes:

- Protease, amylase, lipase and cellulase enzymes work together to quickly dissolve protein, starch and fatty soils in as little as one minute.
- Ideal formulation for your toughest cleaning challenges such as orthopedic and gastrointestinal soils.
- Thorough cleaning for complex instrument surfaces including cannulated areas, box locks and serrations.

Superior Cleaning System:

- Carefully selected surfactants reduce surface tension, improve penetration of cleaning agents and promote free rinsing of dissolved soils.

Multi-Use Detergent:

- Simplify purchasing and inventory. Standardize on one highly effective formula.
- Ideal for high impingement automated washer/ disinfectors and washer/sterilizers.
- No foam formula also indicated for ultrasonic cleaning systems including automated washers for cannulated items.
- Fast, effective cleaning of tough gastrointestinal soils in automated endoscope washers.
- No foam formula improves visibility for increased safety during manual/pan soaking and flushing.

Cost Effective:

- ProEZ AW Quad is highly concentrated with effective dilution starting as low as 1/2 ounce per gallon. Adjust dilution based on soil load and water quality.



ProEZ AW Quad Description	Case Qty	Order#
1 Gallon	4	PREZAW128
1 Gallon Re-Order w/o Pump	4	PREZAWN128
5 Gallon Drum	1	PREZAW050
15 Gallon Drum	1	PREZAW150
30 Gallon Drum	1	PREZAW300
55 Gallon Drum	1	PREZAW550

ProEZ AW Quad™

Concentrated Quadruple Enzyme Automatic Washer Detergent

Description:

ProEZ AW Quad is a four enzyme, non-foaming, neutral pH detergent containing powerful surfactants for use in automatic washers. Protease, amylase, lipase and cellulase enzymes work together with carefully selected surfactants to clean surgical instruments and endoscopes.

Product Specifications:

Dilution Ratios: 1/2 oz. to 1 oz. per gallon.
pH: Neutral.
Temperature of Use: ProEZ AW Quad has an expanded temperature range. It is effective at room temperature about 68°F (20°C). Use with warmer water at 90-110°F (32-43°C) will optimize the performance of the enzymes. This product may be used in automated cleaning equipment (ultrasonics, AEDs, instrument washers) from 90-150°F (32-65°C) according to equipment manufacturer's instructions.

Odor: Fresh.
Color: Blue to green.
Shelf Life: 2 years from date of manufacture.

Intended Use:

ProEZ AW Quad is specially formulated for use in high impingement automated washers and washer/sterilizers. ProEZ AW Quad is intended for cleaning medical instruments by dissolving and lifting organic soils in as little as 1 minute. It is specifically designed for cleaning instruments with difficult to access areas such as endoscope channels, box locks and serrated edges.

Product Capabilities:

ProEZ AW Quad rapidly digests protein, starch and fats, the principle components of surgical soils, and is non-corrosive and safe to use on all instruments.

It is a convenient 4-in-1 product which may be used:

1. As an instrument holding solution.
2. To clean instruments in manual systems.
3. In ultrasonic cleaners and scope reprocessors.
4. In automatic washers.

The non-foaming solution aids in the visualization of instruments when used in manual systems.

Use Precautions:

As with other enzymatic detergents, ProEZ AW Quad is an eye irritant and mild skin irritant. Wear eye protection and water resistant utility gloves when using the product.

Accessory Products:

Other products available from Certol include:



ProEZ foam™
Ready-to-Use Foaming
Enzymatic Spray



ProWash™
Multi-Purpose
Instrument Detergent
and Cart Wash



ProClense™ plus
Instrument Detergent



ProLube Concentrate™
Instrument Lubricant
Concentrate



ProLube RTU™
Instrument Lubricant
Ready-to-Use