


ProSpray™ wipes

Surface Disinfectant/Cleaner Towelettes

"One-Step" Cleaner / Disinfectant / Deodorizer

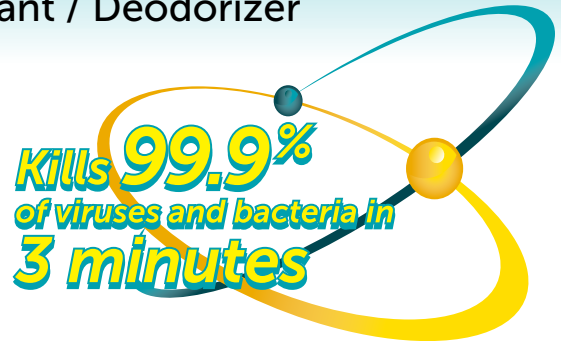

User-Friendly Water Based Formula:

Hard on Germs, Not on Hands or Equipment!

- So safe it is EPA approved for use without gloves (in absence of biohazards).
- Reduces exposure to damaging solvents present in other disinfectants.
- Alcohol-free, ammonium-free and bleach-free actives are best choice for plastics, vinyls and metals.
- One product provides effective cleaning action and disinfecting power.
- Light lemon scent and deodorizer are more pleasant for healthcare workers and patients.

Convenient and Pleasant to Use:

- Packaging offers 40 to 100% more per container compared to leading disinfectant brands.
- Canister and deep well lid designed for easy dispensing of wipes.
- Low profile soft pack is easy to transport and store in drawers.
- Label graphics assist compliance with a prominent contact time and clear instructions for use.



FAST DISINFECTING POWER:

THREE minutes to kill TB and other organisms

Broad Spectrum Disinfecting Power:

- Bactericidal, Tuberculocidal, Virucidal*, Fungicidal.
- Intermediate-level disinfectant meets CDC Guidelines and the OSHA Bloodborne Pathogens Standard for disinfection in the presence of blood and other soils.
- Tested to kill:
 - M. tuberculosis var. bovis (TB)**
 - Staphylococcus aureus**
 - MDROs such as staphylococcus MRSA - HA**
 - Salmonella enterica**
 - Pseudomonas aeruginosa**
 - Fungi (Trichophyton mentagrophytes)**
 - HIV-1***
 - Herpes simplex virus types 1 & 2***

ProSpray wipes Description	Case Qty	Order#
Soft Pack - 9" x 10" - 72/soft pack	12	PSW
Canister - 6" x 6.75" - 240/canister	12	PSWC
Tall Canister - 8.5" x 12" - 135/canister	12	PSWCT
Canister Bracket (fits both regular and large canisters)	1	PSWCR

ProSpray™ wipes

Surface Disinfectant/Cleaner Towelettes

Description:

ProSpray wipes are ready-to-use presaturated towelettes with broad spectrum antimicrobial disinfectant and cleaning action.

Intended Use:

ProSpray wipes are a tuberculocidal, intermediate level disinfectant meeting CDC Guidelines and the OSHA Bloodborne Pathogens Standard for cleaning and disinfecting when blood and body soils may be present. They are intended for use in healthcare and other settings. In the absence of visible soils this product may be used as a "one-step" cleaner-disinfectant.

Product Specifications:

Active Agents: 0.647% o-phenylphenol
0.070% o-benzyl-p-chlorophenol
EPA Reg. #: 46851-12
DIN #: 02419904
Solution Color: Natural color
Odor: Lemon scent
Shelf Life: 2 years from date of manufacture

Product Capabilities:

Kills a broad range of microorganisms in three minutes: Mycobacterium tuberculosis var. bovis (TB), Staphylococcus aureus, Staphylococcus aureus - MRSA HA, Salmonella enterica, Pseudomonas aeruginosa, Trichophyton mentagrophytes (Fungus), HIV-1* and Herpes simplex virus types 1 & 2*.

Note 1: The CDC Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008 continues to emphasize use of intermediate-level disinfectants: "...these surfaces should be disinfected between patients with an intermediate-disinfectant (i.e., EPA-registered disinfectant with tuberculocidal claim)". Also: "Potency against M. tuberculosis has been recognized as a substantial benchmark. Any germicide with a tuberculocidal claim on the label is considered capable of inactivating a broad spectrum of pathogens, including such less resistant organisms as bloodborne pathogens (e.g., Hepatitis B virus, Hepatitis C virus and HIV)." CDC Guidelines for Infection Control in Dental Healthcare Settings - 2003. MMWR 2003;52(No.RR-17): p. 63.

Cleans and Disinfects:

Effective disinfection relies on effective removal of soil. Bloodborne viruses and other dangerous organisms stay viable longer in residual surface soils. ProSpray wipes are ideal for both steps with no need for separate products. Surfactants in ProSpray wipes effectively remove blood and residual soils from a variety of surfaces.

Safer for Surfaces:

ProSpray wipes are a water based disinfectant that meets equipment manufacturers' guidelines for a formula free of alcohol, ammonium and bleach actives. ProSpray wipes will not crack, haze or corrode surfaces commonly found in healthcare settings such as vinyls, plastics and metal fittings. ProSpray wipes also reduce exposure of staff and equipment to solvents used in many disinfectant products. Solvents may boost antimicrobial action but cause equipment damage and chemical sensitivity.

ProSpray wipes are ideal for disinfection of electronic devices, keyboards, control membranes, sensors and cords. The wipes format reduces potential damage from excess liquids seeping into electronic fittings. Consult manufacturer's instructions for specific applications and precautions.

Use Precautions:

A slight soap film may occur upon repeated application of ProSpray wipes due to detergents in the formula. Simply wipe surfaces off at least weekly with plain tap water and paper towels.

Rinse garments or surfaces where ProSpray wipes are used prior to application of any bleach product.

Store ProSpray wipes away from direct heat sources and direct sunlight. Avoid freezing during shipping or storage.

Accessory Products:

Other products available from Certol include:



ProSpray C-60™
Concentrated Surface
Disinfectant/Cleaner



ProSpray™
Ready-to-Use Surface
Disinfectant/Cleaner



ProSpray™ wipes
Canister Bracket
Fits All Certol Canisters



Good Vibrations™
Ultrasonic Cleaner
Solution